GUARDIAN[®]

User Instructions & Warranty | Mode d'emploi
et garantie | Instrucciones de uso y garantía

**Basic Steel Rollator with
8" (20.32 cm) Wheels
Rolling Walker**

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Safety Instructions

To ensure your safety in using the Medline rollator, the following safety information and all instructions must be followed.

General Cautions

- ⚠ **DO NOT use this product without first reading and understanding the Instructions contained in this manual.** If you are unable to understand the warnings, cautions, or instructions, contact a healthcare professional, dealer or technical personnel before use. Failure to do this may result in bodily injury or damage to the device.
- ⚠ The rollator is **NOT** intended to be used to move around while seated.
- ⚠ Rollators are intended for individual use only and are **NOT TO BE USED AS A WHEELCHAIR**. Serious injury to the user and/or damage to the rollator's frame or wheels may result from improper use.
- ⚠ **DO NOT** use the rollator to walk backwards, down gradients or to climb stairs, curbs, or to go over obstacles. Serious risk of fall or injury may occur.
- ✓ The backrest is intended to provide back support while seated. The backrest is not intended to support full body weight. Avoid leaning while sitting as this may result in fall.
- ✓ All wheels must be in contact with the floor and/or ground at **ALL TIMES**. This will ensure the rollator is properly balanced.
- ⚠ **DO NOT** exceed the maximum weight capacity (including basket contents) or serious injury could result. **Maximum weight capacity: 300 lb (136 kg).**
- ✓ Items should be placed completely in bag and should not protrude. Excess weight may cause the rollator to tip, resulting in bodily injury or rollator damage.
- ✓ The basket located under the seat has a weight limitation of 5 lb (2.26 kg).
- ✓ Before each use make sure the rollator frame is fully opened.
- ✓ Periodically wipe down the rollator with a soft cloth and nonabrasive cleaner.

Seat Cautions

- ✓ The brakes **MUST** be in the locked position **BEFORE** sitting on the seat.
- ⚠ **DO NOT** sit on the rollator while it is being loaded into a vehicle or while it is being transported in a vehicle.
- ⚠ **DO NOT** use the seat to transport people or objects.
- ⚠ **DO NOT** sit on the seat when the rollator is on an incline.

Brake Cautions

- ⚠ Check the brakes for proper operation. Make sure that all parts are secure and that the moving parts are in good working order.
- ⚠ Wheels should roll easily and not wobble.
- ⚠ When rising from a lower position, **DO NOT** use the rollator to provide assistance to the user. If the rollator is used to pull the user up to a standing position, the rollator may move even if the brakes are in the locked position and this could result in a fall.
- ⚠ **DO NOT** push the rollator while brakes are enabled. Brakes are to be used to keep the wheels from rolling. However, it may not prevent the rollator from moving if it is pushed.

Note: Only use accessories and spare parts authorized by Medline Industries, LP.

Rollator Parts List

Note You may wish to request a set up and operating demonstration with your local Medical Equipment provider or pharmacy.

Rollator Parts List

- 1 Rollator Frame with wheels
- 1 Backrest
- 2 Handlebars with Brake
- 1 Basket
- 2 Height Adjustment Levers
- 2 Screws and Washers

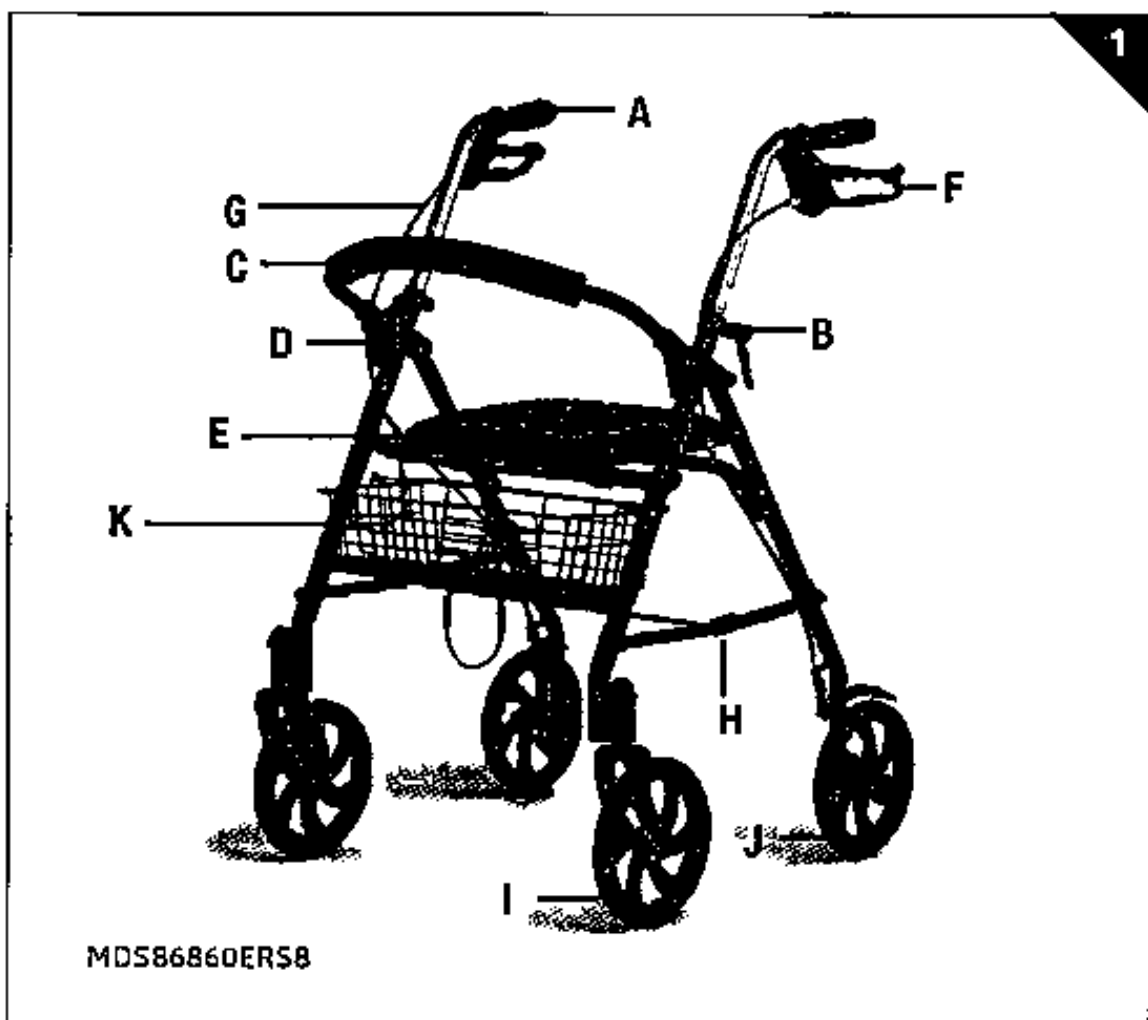
Figure 1: Assembled Rollator

Assembled Rollator

- A. Handgrips
- B. Adjustment Levers
- C. Backrest
- D. Backrest Attachment
- E. Seat
- F. Brake Handles
- G. Brake Cables
- H. Folding Mechanism
- I. 8" (20.3 cm) Front Wheels
- J. 8" (20.3 cm) Back Wheels
- K. Basket

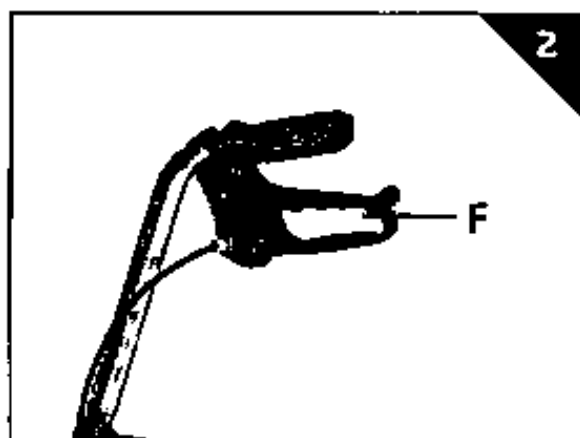
Figure 2: Brake Handle

Assembly Instructions

1. Carefully remove the rollator components from the shipping carton. Refer to diagram (**Figure 1**) as needed.
2. Examine all parts for shipping damage. **DO NOT** use the rollator if there is apparent damage or if any parts (other than the bag) are missing. If there are missing or damaged parts, please call Medline Customer Care toll free at **1-800-MEDLINE**.
3. Remove plastic cover from wheels. Check all of the wheels to see that they are in good working order and roll freely. Please also ensure that all four wheels touch the ground.
4. Unfold the rollator by pushing down on the Folding Bracket (**H**) until the Frame is in full extension.
5. Insert the backrest (**C**) into the attachment pieces so that the curved portion faces the front of the unit. The backrest slides easily into plastic attachment piece.
6. Insert the handlebars so that the handgrips (**A**) are aligned with the rear wheels. The brake cables should be on the outside of the handlebars and frame. Ensure excess brake cable loops towards the front of the rollator.
7. Secure the handlebars by inserting the screw from the inside of the frame through the holes. Align the bolt head so it fits flush against the frame. Apply pressure to the bolt and add the washer on the opposite side of the bolt. Then apply the Height adjustment lever (**B**) and slightly tighten. Adjust the height of the handgrips (**A**) for proper fit.

Determine Proper Height

Stand upright behind the rollator with shoulders relaxed, arms hanging at your side and eyes looking straight ahead. Standing in this position, the handgrips (**A**) of the rollator should be adjusted to a height even with the wrists. Once the handlebar is at the desired height, fully tighten the height adjustment lever (**B**). **Important:** Hand tighten only! The button on the lever can be pushed to rotate the handle to desired angle. Repeat on other side. Make sure the push handles are even and fully tightened before each use.

Operating Instructions

Using the rollator as a walker

For proper positioning and use of the rollator, imagine a line drawn across the back of the rear wheels. Part of one foot should be slightly ahead of that line as you walk.

To Operate the Brakes (Figure 3)

To slow down, squeeze the brake handles up

- To lock the brakes, push down on the brake handles until you feel or hear the brakes click into the locked position. If the brakes have been adjusted but the rear wheels on the rollator move while the brakes are in the locked position, **DO NOT** use the rollator. Please call our toll-free Customer Service number at 1-800-MEDLINE for returns or repairs.

- To release the brakes from the locked position, simply pull up on the brake handles.

Figure 3

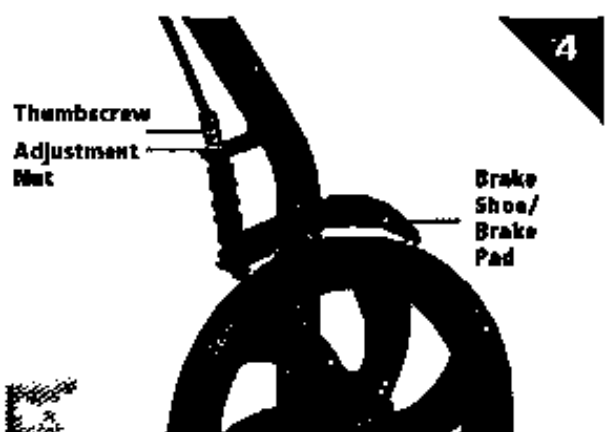
Figure 4

Using the rollator as a seat

Push the rollator to the desired location.

- **LOCK** the brakes **BEFORE** sitting.
- **DO NOT** self propel the rollator while seated.

Brake Adjustment (Figure 4)

1. Loosen the cable nut.
2. Hold brake down onto wheel.
3. Pull down on brake cord.
4. Retighten cable nut.

CAUTION: After making any adjustments to the rollator brakes, you must test the brakes before the rollator is used. See Maintenance Instructions.

Maintenance

Your rollator should be checked periodically to ensure the brakes are functioning properly and that all nuts and bolts are secure.

Test the brakes prior to each use by both walking and squeezing the brakes and by stopping, pushing down on the brakes to a locked position, and then trying to push the walker. If the brakes do not lock the wheels, the brakes need to be adjusted. If adjustment does not tighten the brakes, **DO NOT** use the rollator.

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Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. Non-durable components such as accessories, plastic components, casters and grips are subject to normal wear and replacement. Non-durable components are warranted for 6 months and the frame has a limited lifetime warranty.

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Medline Industries, LP, Three Lakes Drive,

Northfield, IL 60093 USA

Medline United States

1-800-MEDLINE (633 5463)

medline.com | info@medline.com

Medline Canada

1-800-268 2848 | medline.ca | canada@medline.com

Medline México

01-800-831-0898 | medlinemexico.com | mexico@medline.com

Follow us | Suivez-nous | Síguenos

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2022 Medline Industries, LP. Medline and Guardian are registered trademarks of Medline Industries, LP.

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©2022 Medline Industries, LP. All rights reserved./Tous droits réservés./Todos los derechos reservados. Medline is a registered trademark of/est une marque déposée de/es una marca comercial registrada de Medline Industries, LP, Three Lakes Drive, Northfield, IL 60093 USA.